

ONE STEP HIV 1/2 WHOLE BLOOD

Wondfo One Step HIV 1/2 Whole Blood/Serum/Plasma Test Cassette (2 lines) is a rapid one step test for the qualitative detection of HIV1/2 antibodies in human whole blood, serum or plasma at specified cut-off level.


WHO PQ

GENERAL INFORMATION


- Immunochromatography based one-step in vitro diagnostic test
- No instruments needed
- Applied highly purified HIV recombinant proteins: p24, gp41 and gp36
- Detect HIV type 1 and type 2 with separate test lines at one time
- Sensitivity: 99.9%, Specificity: 99.7%
- Specimen: Serum, Plasma or Whole Blood
- Reading Time: 5~20 min
- Storage: 1~30°C



TEST PROCEDURE

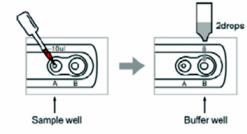

READ THE RESULTS

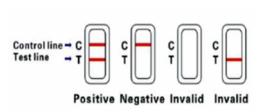

PRECAUTIONS:

- 1. This kit is for in vitro use only. Do not swallow.
- 2. Do not drink, eat, or smoke in areas where specimens are being handled.
- 3. Remove dentures before collecting the specimen.
- 4. All specimens should be treated as potential infectious diseases. Protection glove should be worn when handling the specimen. If gloves come in contact with specimen, change gloves to avoid cross-contamination. Take care of the wastes discarded, such as the test device and buffer solution.
- 5. Icteric, lipemic, hemolysed, heat treated and contaminated blood may cause erroneous results.
- 6. Discard after first use. The test cannot be used more than once.
- 7. Do not use test kit beyond the expiration date.
- 8. Do not use the kit if the pouch is punctured or not well sealed.
- 9. Keep out of the reach of children.
- 10. DISPOSAL OF THE DIAGNOSTIC: The used-device has the infectious risk. The process of disposing the diagnostic must follow the local infectious disposal law or laboratory regulation.

