

FOB (FECAL OCCULT BLOOD) RAPID TEST

FOB (Fecal Occult Blood) is the trace amount of blood in fecal which can not be seen with eyes. It could be the early clinical symptom of the common malignant tumor, the gastric cancer or colonic cancer, or the indicator or enteron ulcer. An early detection of FOB is vital in life saving for it can be checkup out in the early stage of the diseases.

CHARACTERISTICS:

- 1. Accurate: Double antibody sandwich method, high sensitivity.
- 2. Rapid: Get results in 10 minutes.
- 3. Convenient: No need professional training, test could be performed at any time, any place.

Format: Cassette Specimen: Feces Test Time: 10 minutes



INTENDED USE:

Wondfo One Step Fecal Occult Blood Test Cassette is a simple direct binding immunoassay for the rapid, qualitative detection of human occult blood in feces. The assay is designed to detect lower levels of colorectal bleeding than other biochemical methods.


For in vitro diagnostic use only.

PRECAUTIONS:

- 1. This kit is for in vitro use only.
- 2. Wear protective gloves while handling specimens and wash hands thoroughly afterward.
- 3. Avoid splashing or aerosol formation while collecting the sample.
- 4. Discard after first use. The test cannot be used more than once.
- 5. Do not use test kit beyond the expiration date.
- 6. Do not use the kit if the pouch is punctured or not well sealed.
- 7. Keep out of the reach of children.
- 8. Do not read after 30 minutes.

PARAMETERS	CUT-OFF(ng/ml)
Fecal Occult Blood (FOB)	50

TEST PROCEDURE

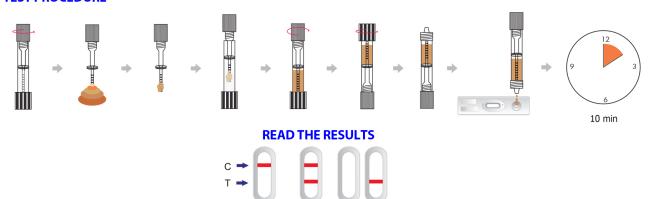



Negative

Invalid

Positive